## **Product Instructions**



## SPTX-PIR v3.0

## **Portable Movement Detector**

- A passive monitors which communicates directly to a Scope Portable Pager.
- Typically the unit is used to detect a person moving within a room or down a corridor.
- The lens is masked so that a narrow beam is projected from the middle of the lens making the monitoring of movement more precise.



## **Instructions for Use**

- 1. Attach the PIR to the wall in the most appropriate position, where the person will move across the beam, with the supplied Velcro pad (use blue tack to determine the best location).
- 2. **SLIDE** the ON/OFF switch down ON ,(DO NOT PUSH the switch IN) the Pager will activate (Range up to 30 50 m. dependant on building construction)
- 3. Press the accept button on the Pager.
- 4. The PIR will reset after NO MOVEMENT is detected for a period of 3 minutes exactly.
- 5. When movement is detected, an alarm call is generated on the appropriate Pagers.
- 6. Go to the room and deal with the resident.
- 7. The PIR2 should be switched OFF when the bed or room is not occupied.
- 8. Position the PIR carefully if being used in a bedroom, too close to the bed and a hand over the side of the bed can activate the system, too far from the bed and the resident can be out of bed and standing before the beam activates.
- 9. Do not position the unit in direct sunlight nor above a radiator.

Lithium Battery PP3 9 volt

approx. 1 year battery life

